

# COVID-19 Ag

Microfluidic Qualitative Immunoassay

The **FREND™ COVID-19 Ag** is an easy-to-use fluorescence immunoassay (FIA) using the FREND™ System designed for the qualitative detection of the **nucleocapsid protein of SARS-CoV-2** directly from individuals suspected with COVID-19. The FREND™ COVID-19 Ag demonstrates reliable results in 3~4 minutes with **one drop or 35 µL** of sample. No complicated process or training is required, thereby improving workflow efficiency.

**Fast**







**Easy**



**Monitoring**



## Steps

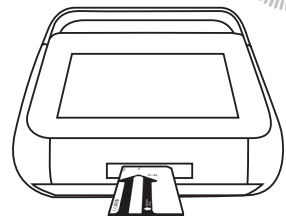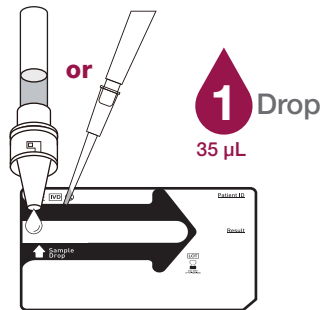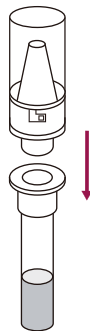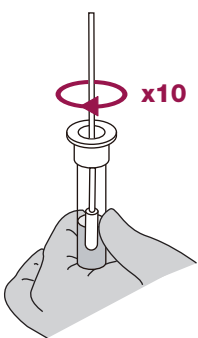

**STEP 01.**

Rotate a swab specimen in a lysis tube and attach a filter dropper.


**STEP 02.**

Add 1 drop (dropper) or 35 µL (pipette) of the sample.

**STEP 03.**

Insert the cartridge and get results.

## Laboratory Information System Connectivity

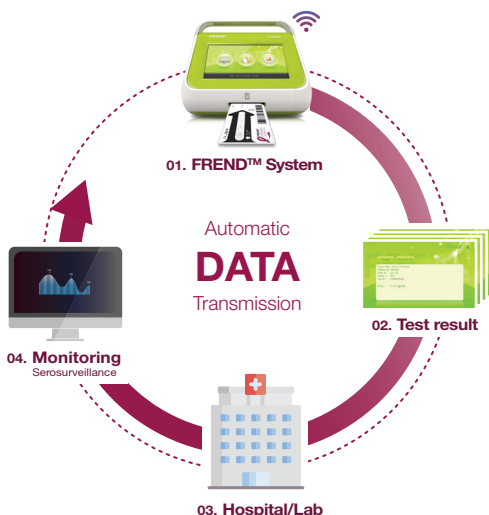

In the event of a pandemic, the management of a vast amount of clinical results is important. As many laboratories face challenges in effective arrangement of the essential information<sup>(1)</sup>, the **FREND™ System** provides the following advantages:

- **Paperless**
- **Time-saving**
- **Automatic Data Transfer & Collection**
- **Convenient Data Management**
- **Can help monitoring (serosurveillance)**

# Clinical Performance Evaluation

A total of 109 clinical samples (34 positive and 75 negative) confirmed with RT-PCR were tested with the FREN<sup>TM</sup> COVID-19 Ag of which 94.12% PPA\* and 100.00% NPA\* were demonstrated.


Reference Method: RT-PCR		
FREN <sup>TM</sup> COVID-19 Ag	PPA*	94.12% (32/34)
	NPA*	100.00% (75/75)



\*Positive Percent Agreement (PPA), Negative Percent Agreement (NPA)

In the test conducted at **S Hospital, Italy**, with a total of 100 samples (60 positive and 40 negative) confirmed with RT-PCR, the FREN<sup>TM</sup> COVID-19 Ag showed a higher performance with 95.00% PPA in comparison to the competitor's with 88.33% PPA.

Reference Method: RT-PCR			
		NanoEntek	Competitor
FREN <sup>TM</sup> COVID-19 Ag	PPA	95.00% (57/60)	88.33% (53/60)
	NPA	100.00% (40/40)	-

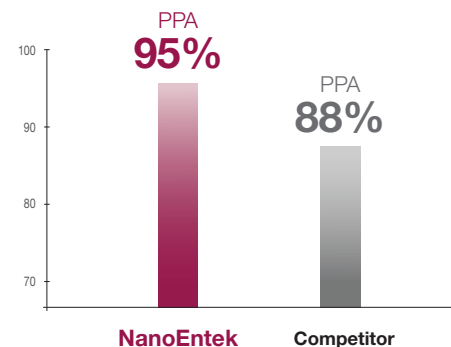

## Specification

Item	Specification
Assay principle	Fluorescence immunoassay
Specimen	Nasopharyngeal swab
Sample volume	1 drop (filter dropper) / 35 $\mu$ L (pipette)
Reaction time	< 3 mins
Quantity	20 tests/kit
Storage condition	2 ~ 30 $^{\circ}$ C (35 ~ 86 $^{\circ}$ F)



## Ordering Information

Product	Cat. No.
FREN <sup>TM</sup> System	F10
FREN <sup>TM</sup> COVID-19 Ag	FRCOG 020P

## Related Products

Cat. No.	Product	Quantity (tests/kit)
EPCO19 100	COVID-19 RT-PCR Kit	100
FRCGFL 020	COVID-19 Ag+FLU A&B	20
FRCOA 020	COVID-19 total Ab**	20
FRCOD 020	COVID-19 IgG/IgM Duo	20
FRCOS 020	COVID-19 SP	20

\*\* FDA EUA Approved



NESCT-FRCOG-001E (V.0.0)

Scan for more information.

## NanoEntek, Inc.

### Head Office

12F, 5, Digital-ro 26-gil, Guro-gu, Seoul, 08389, Korea  
Tel: +82-2-6220-7940/Fax: +82-2-6220-7999

## NanoEntek America, Inc.

Waltham, MA, USA Tel: +1-781-472-2558

## NanoEntek Bio-Technology(Beijing) Ltd.

Fengtai District, Beijing Tel: +86-10-5920-7980

website [www.nanoentek.com](http://www.nanoentek.com)

e-mail [ivdst@nanoentek.com](mailto:ivdst@nanoentek.com)