



Influenza A&B Rapid Card



Total Solution for Influenza A&B Test



Fast Result

Within 3 min. (Max. 8 min.)



Outstanding Sensitivity & Specificity

More than 95%, Diagnosis of Strains Possible



Simple and User-Friendly Test

3 Steps, Soft Tube Squeezed Easily



Remarkable Upgrades for Convenience & Performance



Improved Device Structure
Enhanced Flow,
Preventing Leakage



Applying 3 Drops
Quick Drip, Fast Result




Adoption of Soft Tube
Readily Sampling,
Much Better Usability

BioTracer™


Influenza A&B Rapid Card

Influenza A&B Rapid Card is a product to easily test Influenza A type and B type by using immunochromatography assay.


Test Procedure

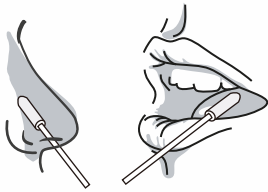


STEP 01
Extract Nasal or Throat secretions with a swab.

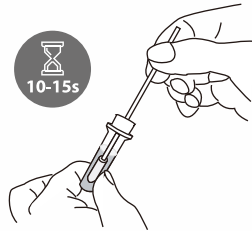

STEP 02
Insert the swab into a collection tube and stir it fully.

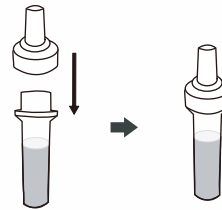


STEP 03
Remove the swab and connect a cap to the tube.

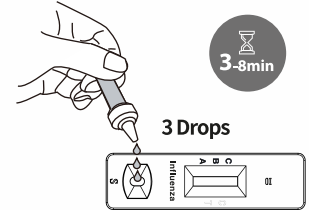


STEP 04
Apply 3 Drops into a sample well and read a result.

Reading and Interpretation of result

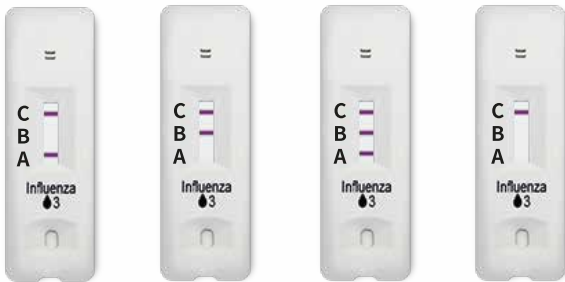


A Type Positive B Type Positive A&B Type Positive Negative

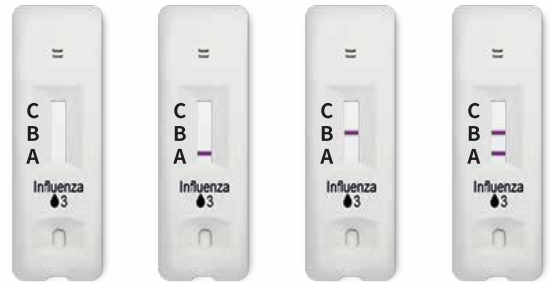

Invalid Result (Re-Test Required)
No Reaction on Control(C) Line

Outstanding Sensitivity & Specificity

Nasal Swab sample	Viral Cell Culture Test			Total
	A Type Positive	B Type Positive	Negative	
A Type Positive	97	-	5	102
B Type Positive	-	94	4	98
Negative	-	-	200	200
Total	97	94	209	400

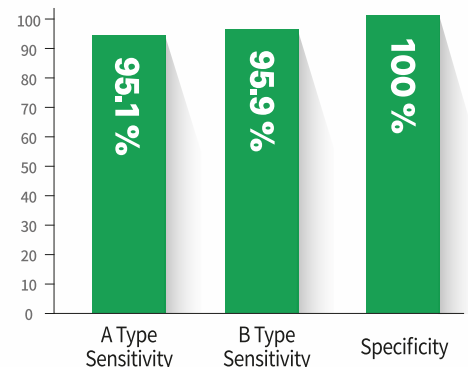

Product information

Cat. No	Product Name	Components	Package	Storage Temperature	Shelf Life
11311	BioTracer™ Influenza A&B Rapid Card	Device, Specimen extraction tube, Filter cap, Specimen collection swab	20 test/kit	2 - 30 °C	24 months